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## Contents

|  |    |
|--|----|
| Disclaimer                             | A5 |
| General introduction                   | A5 |
| Understanding safety and warning signs | A9 |

**Chapter A:  
USER'S MANUAL**

|      |  |     |
|------|--|-----|
| 1    | Intended use   | A13 |
| 2    | Identification                                       | A15 |
| 3    | Description  | A15 |
| 4    | Safety Devices                                       | A23 |
| 5    | Safety instructions for the use of the lift          | A29 |
| §5.1 | Introduction   | A29 |
| §5.2 | General safety instructions                          | A31 |
| §5.3 | Danger zones, risk of crushing & sheering            | A37 |
| §5.4 | Additional instructions for the wheelchair occupants | A41 |
| §5.5 | Loading diagram                                      | A43 |
| §5.6 | Importance of maintenance                            | A47 |
| 6    | Operation of the lift                                | A49 |
| §6.1 | Introduction to the control units                    | A49 |
| §6.2 | Operating instructions                               | A51 |
| §6.3 | Emergency operation                                  | A57 |
| §6.4 | Electric emergency stop                              | A61 |
| 7    | Warranty terms                                       | A61 |

**Annexes:**

|                                    |     |
|------------------------------------|-----|
| Annex 1 • Safety and warning signs | A63 |
|------------------------------------|-----|

The CE-certificate of conformity and the instructions regarding compulsory inspections and reports are included in a separate manual IDENTIFICATION AND INSPECTION BOOK.

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## DISCLAIMER

The illustrations and information contained within this manual are not contractually binding, and cannot lead to any form of legal action against **DHOLLANDIA**.

**DHOLLANDIA** lifts are constantly being adapted to new vehicle and chassis developments, and specialised customer requirements. Therefore **DHOLLANDIA** reserves the right to alter product specifications without prior notice; and potentially modifications or new developments might not have been taken into account at the time of printing.

This manual focuses on the wheelchair application. Where these lifts are used for commercial purposes, certain features (such as safety rails, emergency stops, emergency controls, ...) might have been omitted or demounted.

If you require further information on non-conforming parts, contact your official **DHOLLANDIA** agent for advice.

The illustrations shown in this manual represent lifts prepared for left-hand drive countries (= continental Europe). The operators of lifts prepared for right-hand drive countries (= UK, Ireland, South Africa, Australia,...) shall consider that their equipment, if ordered with the correct option number, is executed mirror wise.

## GENERAL INTRODUCTION

This manual explains :

- chapter A: how the **DHOLLANDIA** lift is manufactured; what safety devices are incorporated in its design; and how to use the lift in a correct way, that guarantees the integrity of the machine over the intended life time and the safety of the operator and any bystanders.
- chapter B: how to maintain and service the lift in an appropriate way, that can guarantee its reliability over the intended life time.

Refer to the separate CE-handbook supplied with the lift for all information re. compulsory testing, test reports, maintenance and repair reports.

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## ⚠ WARNING

- This manual must be kept with the lift at all times, as reference book for the operators and technical service personnel.
- In order to ensure the safety of the operator, the wheelchair occupants and occasional bystanders, the use of the lift is strictly reserved to skilled operators, who have been duly trained, who know and understand the full content of this manual.
- Negligence can put the operator and third parties at great risk.

## ⚠ WARNING

- It is strictly forbidden to modify the construction of the lift and its safety devices in any way.
- **DHOLLANDIA** disclaims liability for any personal injury or property damage that results from operating a product that has been modified from the original design, without explicit written approval from the manufacturer.
- The Declaration of Conformity, the manufacturer's product liability and the warranty are voided in case of disregard in this respect, and when non-original components or spare parts are used to service or repair the lift.

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## UNDERSTANDING SAFETY AND WARNING SIGNS

- A great number of safety signs used in this manual are based on international standards, others are tailor made to refer to specific situations or actions.
  - Consult the annex to chapter A for an overview of signs used, and their meaning.
  - Read and understand these signs in conjunction with the instructions of this manual, to ensure maximum safety for the operator and occasional bystanders. Ignorance or negligence to understand these instructions can put the operator and third parties at great risk.
- Please take special notice of the following signs. They mark the likelihood and severity of a potential injury if a person doesn't follow the instructions contained on the safety sign.

**DANGER:** indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury. [white letters on red background]

**WARNING:** indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. [black letters on orange background]

**CAUTION:** indicates a potentially hazardous situation which, if not avoided, could result in minor or moderate injury. [black letters on yellow background]

**NOTICE:** indicates a potentially hazardous situation which, if not avoided, could result in property damage. [white letters on blue background]

**SAFETY INSTRUCTIONS:** indicate general instructions relative to safe work practices, reminders of proper safety procedures, or the location of safety equipment. [white letters on green background]

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## 1. INTENDED USE

- **DHOLLANDIA commercial lifts** [DH-RB 05.02.....series used for commercial purposes] are designed to be fitted to commercial vehicles (factory built panel vans, commercial trucks, trailers and semi-trailers), and shall be used exclusively to load and unload the goods transported on this vehicle.
- **DHOLLANDIA wheelchair lifts** [DH-RB 05.02..... series] are designed to be fitted to busses and coaches, and shall be used exclusively to assist wheelchair passengers to embark and disembark the vehicle.
- In both cases, the lift shall be used with obedience of the loading diagram, and the user's and safety instructions contained in this manual.

### WARNING

- It is strictly forbidden to use the lift in a different way, or for different purposes, than foreseen in fore lying user's manual.
- Negligence can put the operator and third parties at great risk.
- **DHOLLANDIA** disclaims liability for any personal injury or property damage that results from improper use.
- The manufacturer's product liability and the warranty are voided in case of disregard in this respect.

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## 2. IDENTIFICATION

- Every DHOLLANDIA lift is identified by and labelled with a unique serial number. Use this reference for any inquiry on this particular lift, or when ordering spare parts.
- Apart from the lift type and serial number, the main identification sticker in the control box also mentions the nominal lift capacity, and the date of construction [fig. 2.1].

## 3. DESCRIPTION


DHOLLANDIA lifts are developed and manufactured using state-of-the-art technology, high quality materials and components, and comply with the European CE safety regulations mentioned in the Declaration of Conformity (unless agreed otherwise upon ordering - export outside CE region).

| N° | Description ...  |
|----|------------------|
| a  | Xx.....<br>..... |

In the tables below, the items marked in white on black background count among the “safety devices” described in section 4 below.

- Further details & terminology [fig. 3.1 + following]:

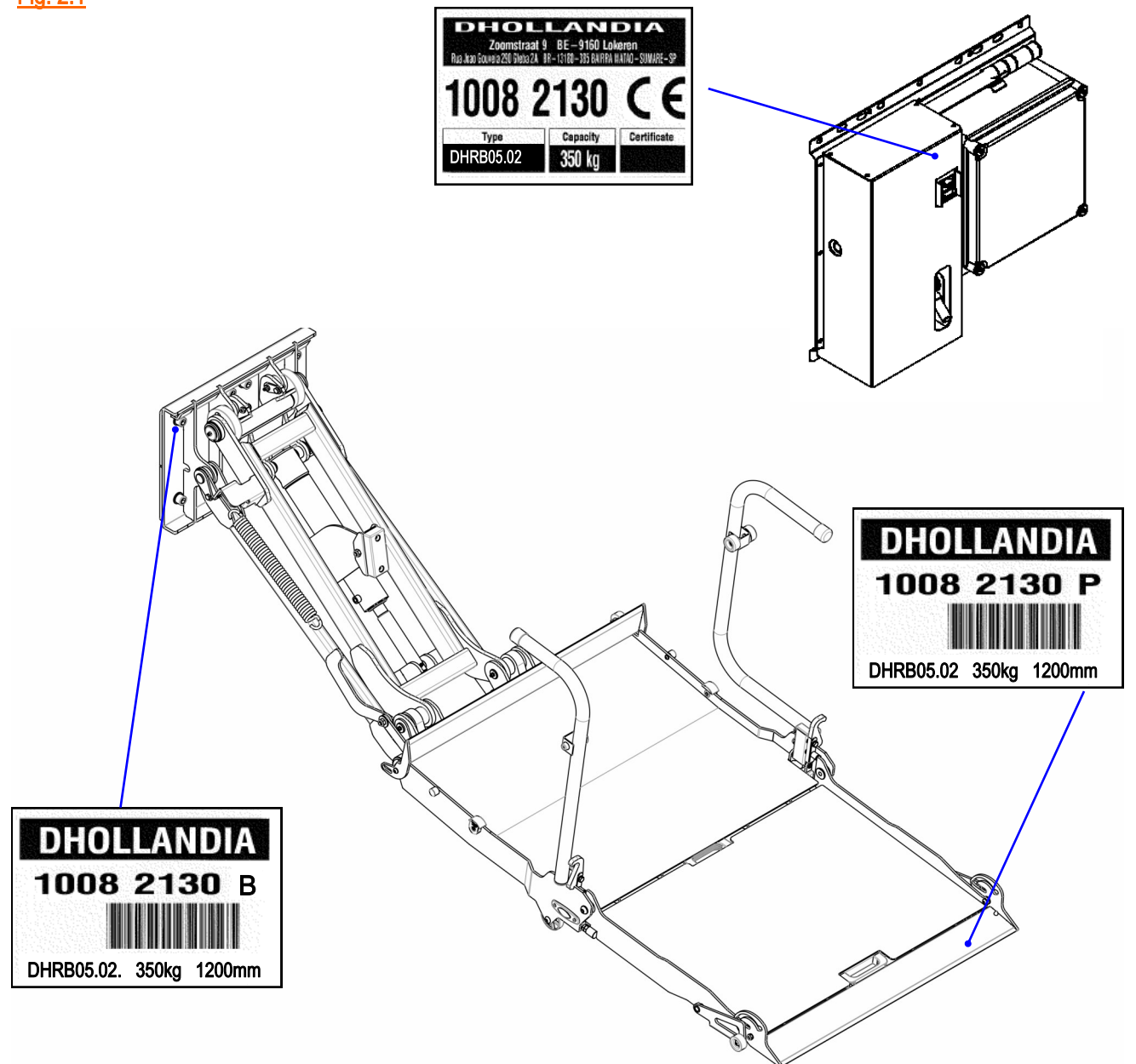Fig. 2.1

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
| N°  | Description   |
|-----|---|
| 1   | <b>Lift arms + lift cylinders</b> , used to lift / lower the platform, the wheelchair passenger and / or operator.  |
| 2   | <b>Platform</b> , carries the load or wheelchair during lifting / lowering. The platform surface incorporates an anti-slip protection profiling.  |
| 3   | <b>Safety rails</b> , to be manually raised and locked in vertical position prior to accessing the platform, and to be restored manually when ending the use. They provide a hand-hold for the platform occupant.   |
| 4   | <b>Automatic roll-stop</b> at the rear edge of the platform. Is lowered automatically when the platform touches the ground. Is raised automatically when the platform is lifted off the ground.<br>Prevents wheeled goods or the wheelchair from slow, unexpected rolling off the platform during the lifting / lowering. |
| 5   | <b>Flashing platform lights</b> , to signal the platform in surrounding traffic.  |
| 6   | <b>Automatic bridge plate</b> to the loading floor of the vehicle.<br>Functions as toe-guard during the lift cycle. Is lowered automatically to horizontal position when the platform arrives at floor level, creating a smooth and safe passage between the platform and the vehicle floor.                              |
| 7   | <b>Outboard edge</b> of the platform.   |
| 8   | <b>Inboard edge</b> of the platform.  |
| 9   | <b>Hydraulic pump unit</b> , steering all functions. Contains the electric motor driving the hydraulic pump and the control valves.   |
| ... | ...   |

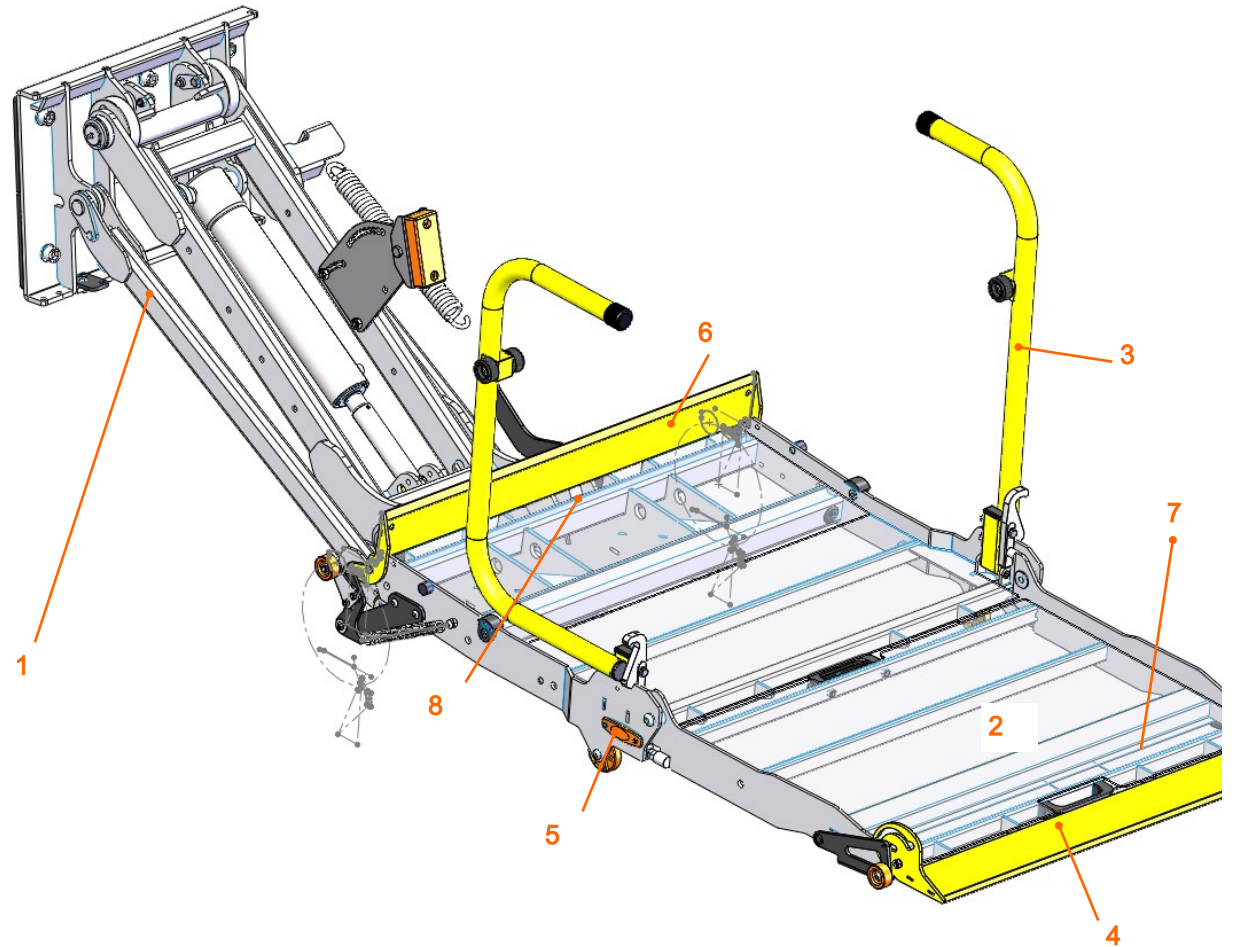Fig. 3.1

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| N°  | Description  |
|-----|--|
| ... | ...  |
| 10  | <b>Electric box</b> , contains the main electric board, the signal LED's (for diagnosis) and the electric emergency buttons (in case of electric malfunction).   |
| 11  | <b>Wander lead with spiral cable</b> , as main control unit for the lift. The control unit can be equipped with an incorporated emergency switch [ref. OWE005.02] or with a separate emergency switch box [ref. OWE005.01] |
| 12  | <b>Electric emergency stop</b> . Allows to switch-off the electric power to the control unit, and to cause an immediate stop of all electrically actuated lift movements.  |
| ... | ...  |

Fig. 3.2

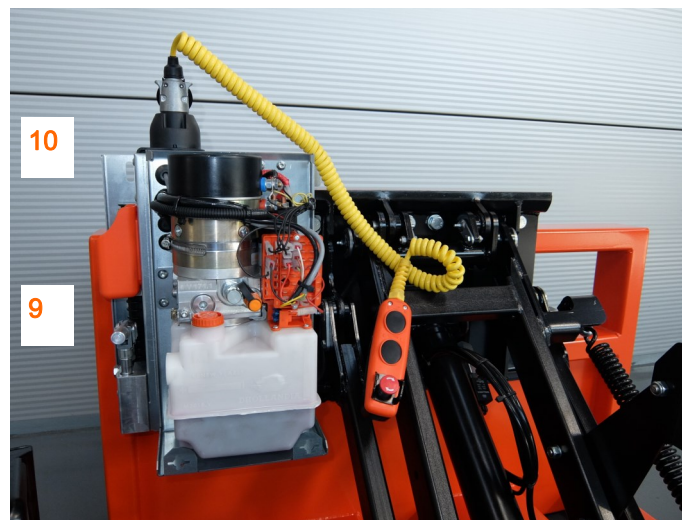
Fig. 3.3

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- For all lift models above, the electric control box and the hydraulic power pack are connected to the vehicle battery (or auxiliary battery) during fitting, according to the fitting instructions of **DHOLLANDIA**, and the requirements imposed by the vehicle manufacturer.

### **NOTICE**

- To ensure the reliability of the lift over many years, it is extremely important that the batteries, their charging system, the battery- and earth cables, and fuses are dimensioned sufficiently strong, and fitted with care following above mentioned instructions. Insufficient battery tension will cause harm and irreparable damage to the electric components of the lift (starter solenoid, electric motor, electric switches, etc...).
  - Vehicle manufacturers often impose prescribed connections for the battery and earth connections of the lift, which should be observed to avoid damage to the vehicle electrics. Where such prescriptions conflict with the fitting instructions of **DHOLLANDIA**, contact the vehicle importer or **DHOLLANDIA** for further advice.
- As most vehicles are supplied with power supplies that cover the primary needs of the original equipment only, the vehicle owner & his contractors should consider upgrading the battery capacity and charging system in function of the additional needs of all auxiliary equipment (lift, but also extra heating system, refrigeration units, etc...).
- In case of long journeys and short periods of lift use, the existing alternator and batteries will probably suffice.
  - In case of long journeys and long periods of intensive lift use, the batteries will probably need upgrading by heavier duty ones.
  - In case of short journeys and long periods of intensive lift use, both the alternator and the batteries will probably need upgrading.

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
#### 4) SAFETY DEVICES

- DHOLLANDIA lifts are equipped with a wide range of safety devices, to enable its intended use [see section 1] with a maximum degree of safety for the operator, the wheelchair passenger and / or the load on the platform.
- The following safety devices are incorporated on the DH-RB 05.02.... Series:

| N°  | Description  |
|-----|--|
| 1   | <b>Anti-slip surface</b> : the platform surface incorporates an anti-slip protection profiling.  |
| 2   | <b>Automatic roll-stop at the outboard edge of the platform.</b> Is lowered automatically when the platform touches the ground. Is raised automatically when the platform is lifted off the ground. Prevents wheeled goods or the wheelchair from slow, unexpected rolling off the platform during the lifting / lowering. |
| 3   | <b>Safety rails,</b> to be manually raised and locked in vertical position prior to accessing the platform, and to be restored manually when ending the use. These provide a hand-hold for the platform occupant.  |
| 4   | <b>Automatic bridge plate</b> to the loading floor of the vehicle. Functions as toe-guard during the lift cycle. Is lowered automatically to horizontal position when the platform arrives at floor level, creating a smooth and safe passage between the platform and the vehicle floor.                                  |
| 5   | <b>Flashing platform lights,</b> to signal the platform in surrounding traffic.  |
| ... | ...  |

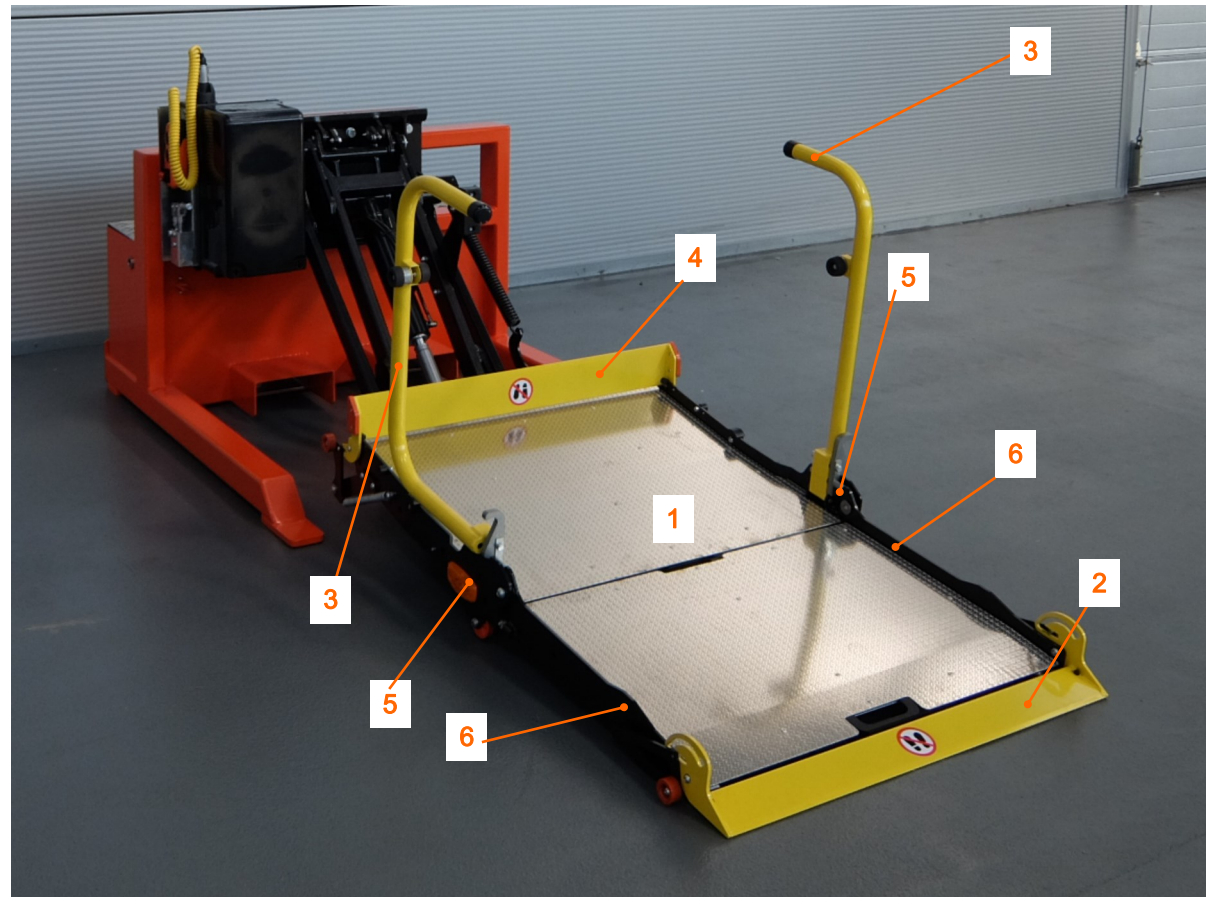**Fig. 4.1**

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
| N°  | Description  |
|-----|--|
| ... | ...  |
| 6   | <b>Outer side skirts</b> , preventing finger traps for accidental bystanders on the sides of the platform, between the different elements of the lift arms.  |
| 7   | <b>Electric emergency stop</b> , Allows to switch-off the electric power to the control unit, and to cause an immediate stop of all electrically actuated lift movements. Is either incorporated in the control unit [ref. OWE005.02] or supplied as a separate box [ref. OWE005.01] |
| 8   | <b>Main battery fuse</b> on the (+) battery cable  |
| 9   | <b>15A fuse</b> on the wires inside the power pack<br><br>Both fuses are used to protect the electrical system against short-circuit and amperage peaks.   |
| 10  | <b>Hydraulic flow control</b> : the main valve in the hydraulic power pack is equipped with a braking valve, retaining the descend speed within legal boundaries: max. 15cm / sec for vertical movements.  |
| 11  | <b>Emergency hand pump</b> : allows to LIFT the platform in case of electrical failure (battery deficiency, damaged fuse, damaged electrical controls...)  |
| 12  | <b>Emergency exit button</b> : allows to LOWER the platform in case of electrical failure (battery deficiency, damaged fuse, damaged electrical controls...)   |
| ... | ...  |

Fig. 4.2

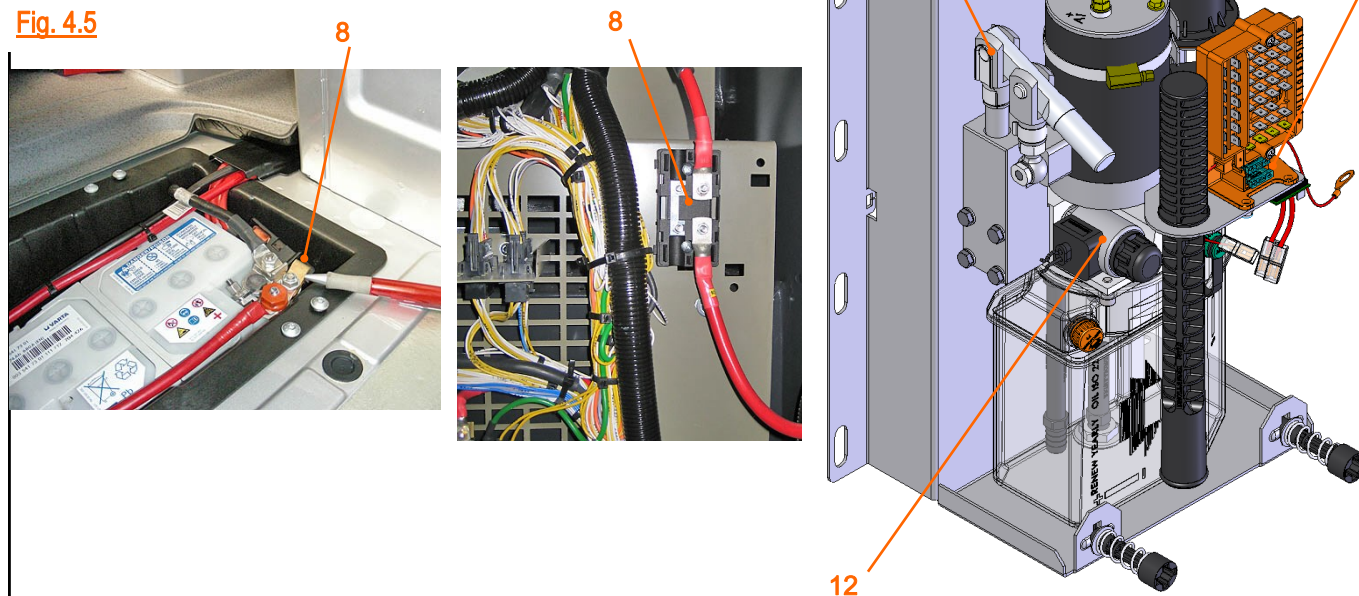Fig. 4.5

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| N°  | Description  |
|-----|--|
| ... | ...  |
| 13  | <b>Load detection / pressure relief valve</b> : permits the manufacturer and the installer to limit the final performance of the lift to its nominal capacity, and protects it against overload during the LIFT cycle.   |
| 14  | <b>Identification stickers</b> : mean to identify the serial number of the lift (needed to order spare parts, or inquire for other information).<br><br><b>Warning decals</b> : mean to warn to the operator and the wheelchair passenger for specific hazards.          |
| 15  | <b>Platform visibility and marking</b> : if, in operating position, the platform protrudes beyond the extremities of the vehicle, warning devices shall be fitted so that it can be seen from all approachable sides in the daylight and at night (whatever applicable). |

Fig. 4.6

Fig. 4.7

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## 5) SAFETY INSTRUCTIONS FOR THE USE OF THE LIFT

### §5.1 - Introduction

#### **⚠ WARNING**

- In order to ensure the safety of the operator, the wheelchair occupants, and occasional bystanders, the use of the lift is strictly reserved to skilled operators, who have been duly trained, who know and understand the full content of this manual and conscientiously apply the safety instructions given in this manual.
- Negligence can put the operator and third parties at great risk.
- Simultaneously, the operator should follow all other prescriptions and instructions applicable to his work situation (e.g. health & safety regulations, road traffic regulations, company procedures, Driver Periodic Training according to 2003/59/CE, etc...).
- The operator should have passed the minimum age of 18 years.
- These safety instructions are of general nature, applicable for the whole range of lifts. Some images might show deviating products, but the principles apply.

Fig. 5.1.1

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## §5.2 - General safety instructions

### ⚠ WARNING

In order to ensure his own safety, the safety of the wheelchair occupant and occasional bystanders, the operator should comply with all safety instructions in chapter 5 while operating the lift.

- The operator should use appropriate working clothes. Never wear loose-fitting clothes that may be trapped in the moving parts of the lift. Always wear safe footwear with steel protection tips and protective gloves. [fig. 5.2.1]
- Prior to opening the vehicle doors, check if the lift can be used safely. Take precautions to guarantee your own safety, the safety of the platform occupants, and the safety of occasional bystanders and third parties in traffic.
- Clear the working area of any objects that could potentially impede the movements of the lift.
- Make sure the platform is visible from all approach directions (flashing lights, platform flags, traffic cones, etc...), and that the working area is sufficiently illuminated.
- The vehicle must be safely parked with the hand brake on, and the engine must be switched off. Only use the lift if the vehicle is parked on level ground. It is dangerous to use the lift when the vehicle is parked on a slope. [fig. 5.2.2]. Lock the vehicle doors, and fasten all other moving parts of the vehicle body.
- Ensure that the vehicle cannot tip-over when putting heavy weights on the platform.

Fig. 5.2.1



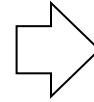
Fig. 5.2.2

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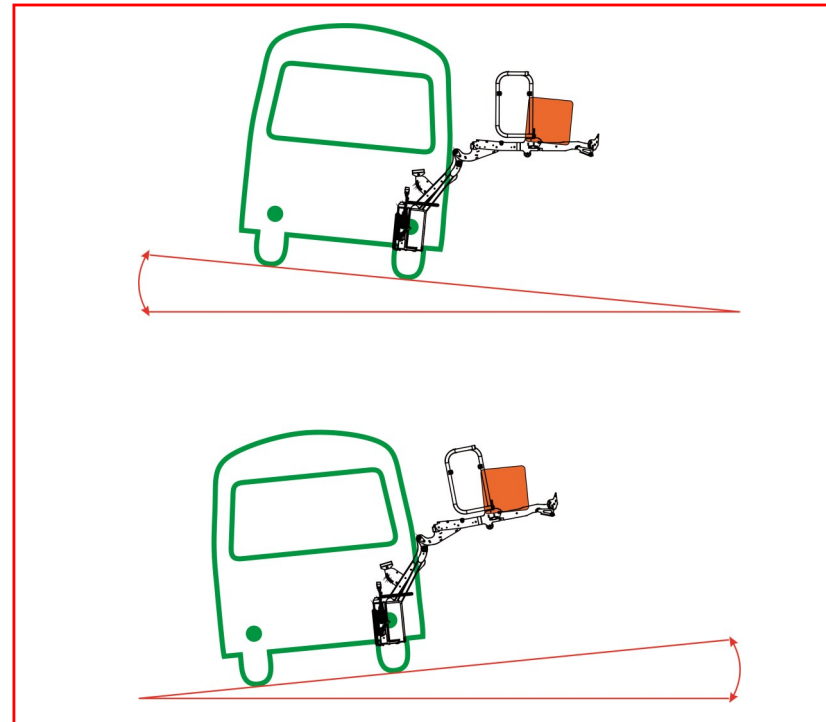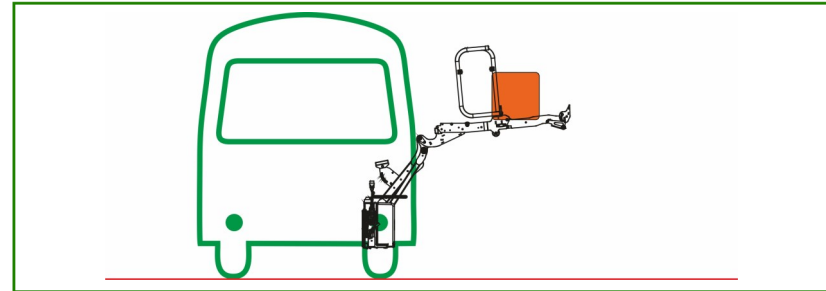- If the vehicle or the lift are equipped with mechanical or hydraulic stabilising legs, deploy these before opening the platform. Ensure that the stabilising legs are positioned on solid even ground. In case of soft terrain (sand, gravel,...), solid support blocks must be used under the stabilising legs [fig. 5.2.3].

- The operator should overview and keep a clear visual control over the whole working area of the lift and its platform, and the wheelchair passenger on the platform at all times.

- The operation of the lift should be confined to a single operator. When opening / closing the lift, the operator should not permit anyone to stand near the moving platform.
- When lifting / lowering, the operator should not permit anybody, but himself and the wheelchair passenger, to be in the working zone of the lift and its platform. Ensure at all times that nobody stands under, behind or within reach of the moving platform.
- Careful supervision is necessary if used on behalf of, or near children.


- Inspect the lift prior to each use. If any unsafe condition exists or unusual noises or movements are noticed, DO NOT use the lift and contact an authorized **DHOLLANDIA** service agent for repair.

- Read and comply with all warning labels and symbols affixed to the lift.
- Respect the nominal capacity and the loading diagram of the lift.

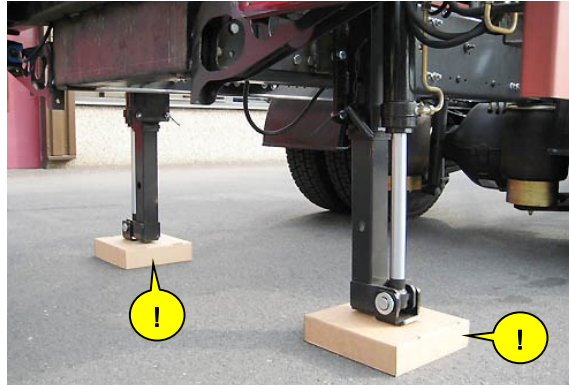Fig. 5.2.3

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- Never leave the lift behind in open position. Before leaving the vehicle unattended, stow the lift away, and close the doors of the vehicle.
- Don't move the vehicle with the lift in open position. Stow the lift away, and close the doors of the vehicle before moving position. Never move the vehicle while a person stands on the platform of the lift



- To prevent injury by slipping, always wear professional safe footwear with a good non-slip sole. If the platform surface is slippery because covered with snow, mud and dirt, or slippery liquids, DO NOT use the lift and clean the platform first. Do not run on a lift platform.

- To prevent injury by tripping, pay attention to protruding items on the platform surface at all times (ex. Side barriers, roll-stop flap, etc...).

- The lift should not be used if the operator is intoxicated or impaired in any way.
- The lift should be used by means of original control units only.
- In traffic, always respect a safety distance of 5m to the following car, and ask other drivers to respect an equivalent distance when parking behind your lift. [fig. 5.2.4]

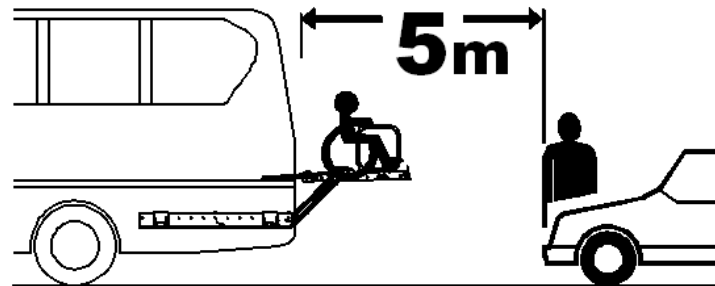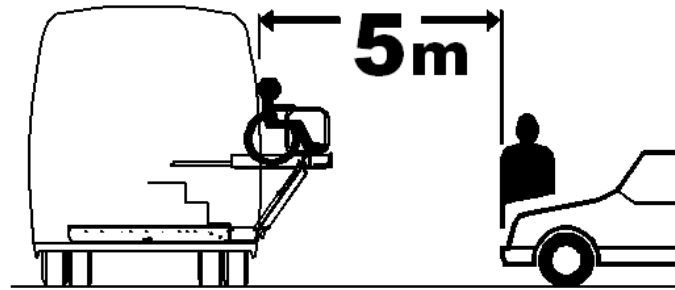Fig. 5.2.4

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### §5.3 - Danger zones, risk of crushing & sheering

⚠ WARNING

- Always stand clear of the platform area
- Lift hazards can result in crushing or falling
- Keep hands and feet clear of pinch points and moving parts
- When riding the lift, make sure the load is stable, and your footing is solid

- Hold arms, feet and clothing well clear of all moving parts of the lift. Beware at all times for potential risks of crushing or sheering fingers or hands, toes or feet between the moving parts of the lift arms, hydraulic cylinders and the moving platform. [fig. 5.3.1 - 5.3.2]. On the platform, the operator and wheelchair passenger should take hold of no other parts but the safety rails foreseen for this purpose.
- Avoid the hazardous crushing zone between the raising platform and the rear frame of the vehicle floor. Ensure that the feet are not squeezed between the platform and the bumper, or any other protruding parts of the vehicle body. [fig. 5.3.1]

Fig. 5.3.1

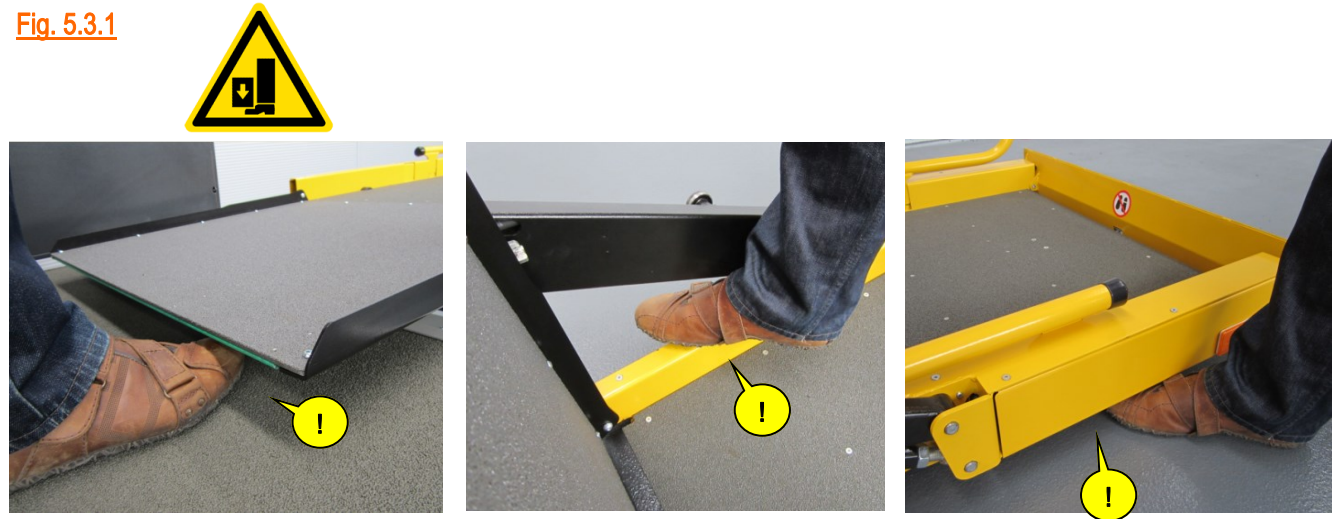
Fig. 5.3.2

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**NOTICE**

- Do not allow anybody to stand on, or hinder the bridge plate at the front edge of the platform. [fig. 5.3.3]
- Do not allow anybody to stand on, or hinder the automatic roll-stop at the rear end of the platform. [fig. 5.3.4]
- The operator and the wheelchair passenger should not leave the platform before it has reached its final parking position, either at ground level, or at the loading floor of the vehicle. Third persons assigned to help the wheelchair passenger embark or disembark, should stay out reach of the platform until its has reached its final parking position, and has come to a complete stop.

Fig. 5.3.3


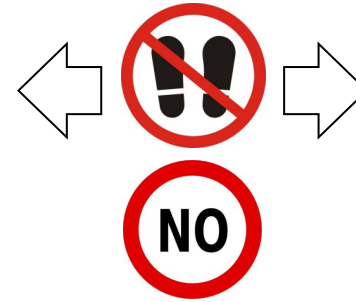
Fig. 5.3.4

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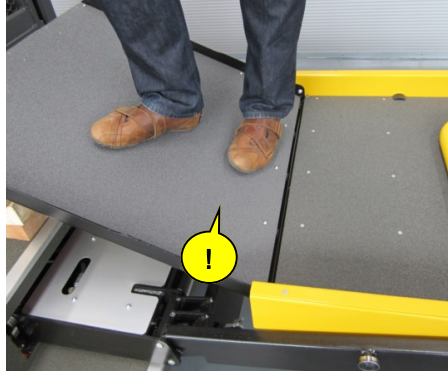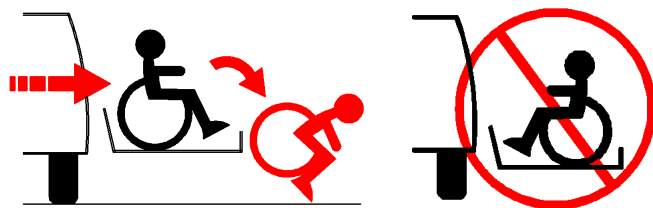
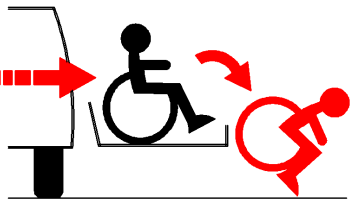
### §5.4 - Additional instructions for the wheelchair occupants


- The roll-stop prevents the wheelchair from SLOW unexpected rolling off the platform during lifting and lowering.
- The roll-stop is NOT designed to bring FAST moving wheelchairs, or the big rear wheels of a wheelchair to a complete stop.
- If possible, electric wheelchairs should be switched off to manual mode before accessing the platform, and be pushed manually onto the platform by the operator.
- It is recommended that wheelchair passengers are lifted / lowered facing outward, with the small front wheels of the wheelchair adjacent to the roll-stop at the rear edge of the platform. [fig. 5.4.1]
- Standees should use the lift facing the direction of travel, toward the vehicle when embarking, or away from the vehicle when disembarking.

**⚠ WARNING**

- Always board platform slowly and cautiously. Keep speed low to avoid falling off the platform.
- Lift and lower wheelchair passengers facing outward.
- On the platform, wheel brakes should be engaged. Electric wheelchairs should switch power off.

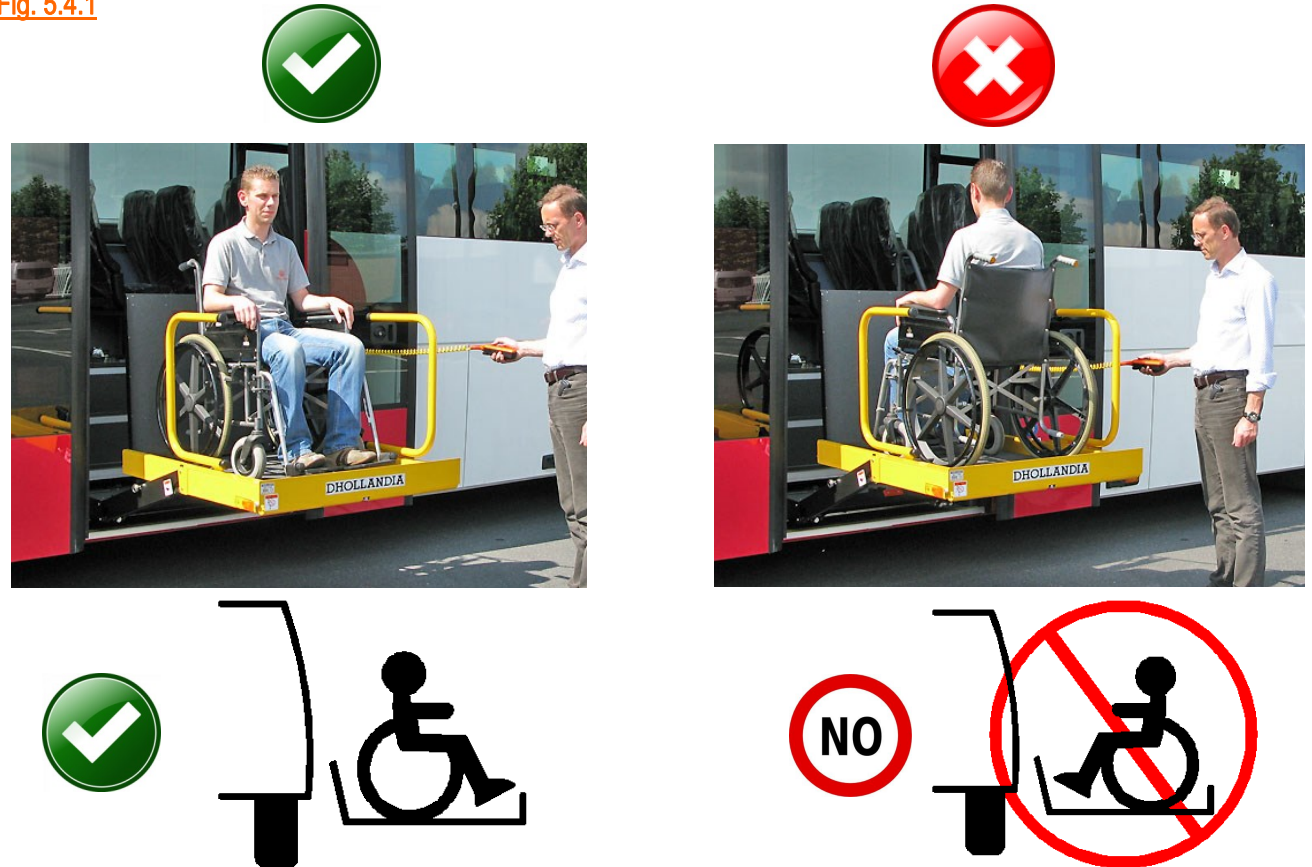Fig. 5.4.1

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- Ensure the wheelchair fits safely on the platform, and doesn't extend over the edges of the platform, doesn't interfere with the automatic roll-stop at the rear, nor the bridge plate at the front of the platform.
- When exiting the vehicle, never back onto the platform. Always face outward, ensure the platform is safely in place at the loading floor of the vehicle, the side safety rails are locked into position, and the automatic roll-stop is up and locked into position.
- Use extreme care in wet or slippery conditions. Wheelchair brakes are less effective if the platform and / or wheels of the wheelchair are wet or dirty.
- Ensure the passengers take a central position between the lift arms. [fig. 5.4.2].

- If available, the platform occupants should grip both side safety gates. [fig. 5.4.3 - 5.4.4]

§5.5 - Loading diagram

**NOTICE**

- Hydraulic lifts are not designed to LIFT / LOWER weights corresponding to their nominal capacity over the full surface of the platform. The nominal capacity is valid at a **centre of gravity of 500 mm** behind the vehicle body. Behind that point, the maximum safe working load diminishes.
- When LIFTING, the lift is normally protected against overload by the pressure relieve valve in the hydraulic circuit. Most of the overload, and consequential damage, happens when LOWERING loads.

Fig. 5.4.2


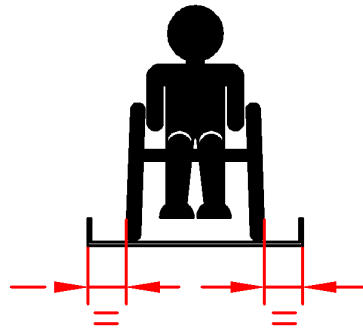
Fig. 5.4.3

Fig. 5.4.4

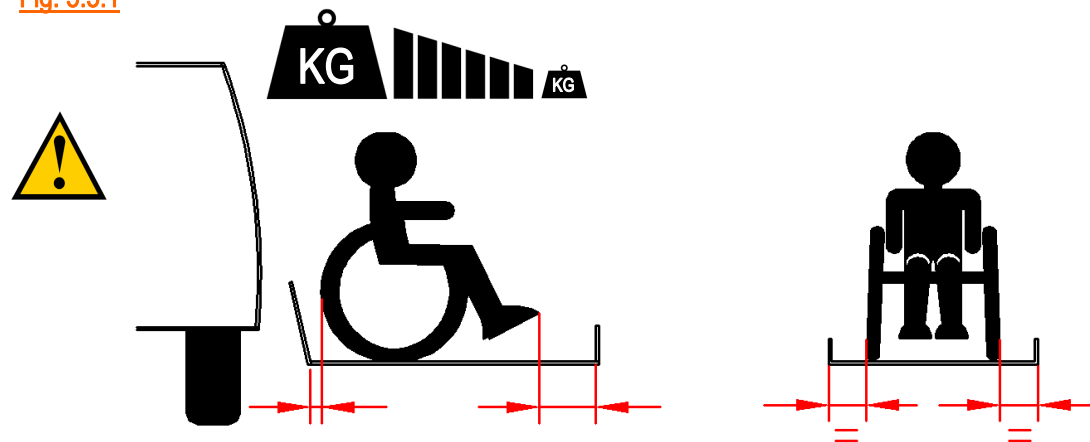Fig. 5.5.1

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
- In order to prevent premature wear or damage of the lift, possibly resulting in accidents with injuries, it is important to follow the instructions below.
- The **nominal capacity** is indicated on the identification sticker and on the CE certificate of conformity [fig. 5.5.2]. It is the maximum weight allowed to be carried on the platform in the best possible circumstances. This is under the following conditions [fig. 5.5.1]:
  1. In longitudinal direction: the load or wheelchair is placed as close as possible to the floor of the vehicle
  2. In lateral direction: the load or wheelchair is placed at a central position between the lift arms (not out of centre to one of the sides of the platform).
- Whenever the loading circumstances deviate from these best possible conditions, the **safe working load (=SWL)** decreases by 30%.
- The dimensions of the wheelchair should never exceed those of the platform. The wheelchair should never protrude beyond the perimeter of the platform.

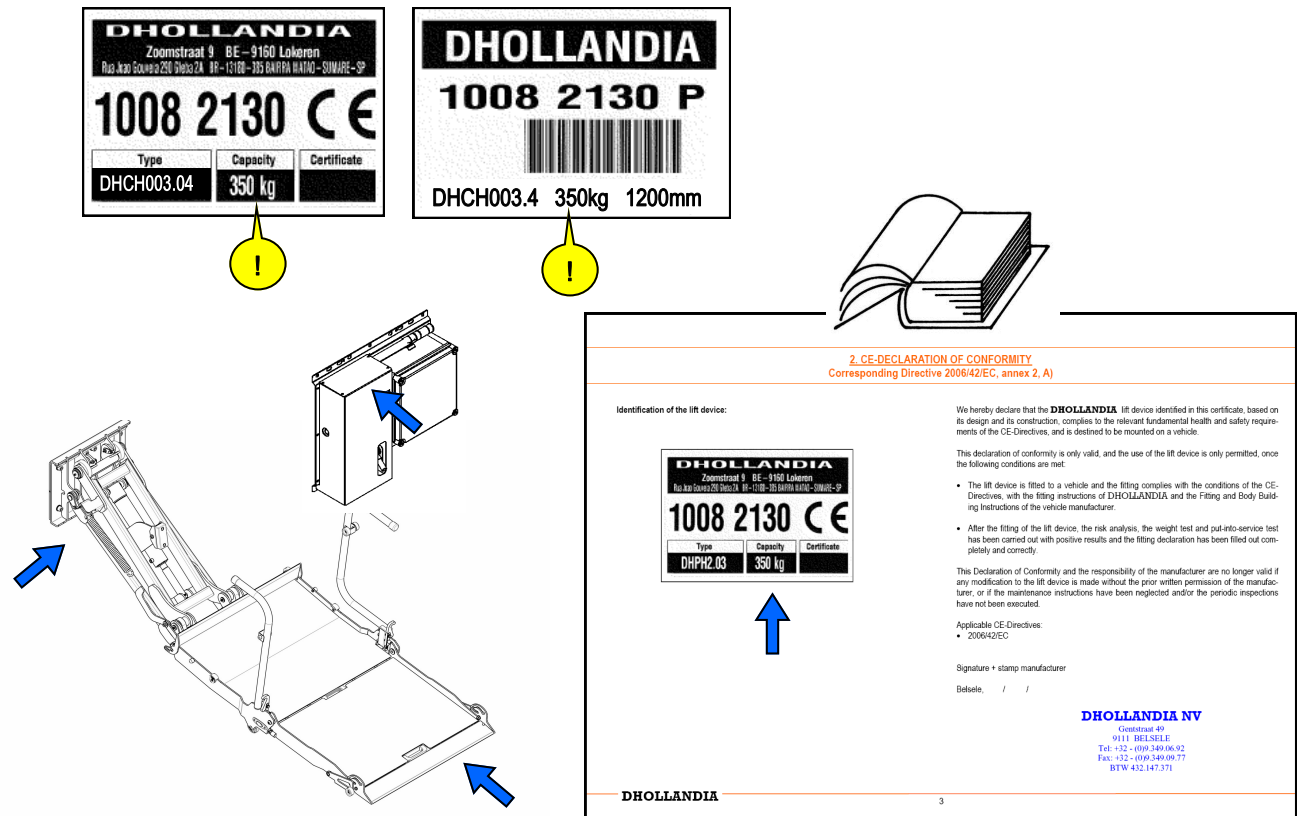Fig. 5.5.2

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### §5.6 - Importance of maintenance

#### **⚠ WARNING**

- Prior to use, inspect the lift to ensure that all safety systems and all functions operate correctly, and that no urgent maintenance or damage repair is required. If any unsafe condition exists or unusual noises or movements are noticed, DO NOT use the lift and contact an authorized **DHOLLANDIA** service agent for repair.
- Missing, worn or illegible warning decals must be immediately replaced.

#### **NOTICE**

- Competent and regular maintenance is very important for the operational reliability, and the safety of the operator and occasional third parties in traffic.
- All maintenance and repair works should be executed by authorized **DHOLLANDIA** service agents, and using original spare parts only.

#### **⚠ WARNING**

- If a lift cannot be repaired immediately in case of breakdown, it must be put out of operation and secured against unauthorized use.
- A lift put into its travel position with the help of external devices (forklift, crane,...), is not supported by its hydraulic cylinders. After releasing the mechanical platform lock, the platform and lifting mechanism will drop in free fall to the start point of the breakdown, without any possibility to stop the fall by means of the regular controls. This can result in the unaware operator and third parties being at great risk. [See Fig. 5.24]
- To avoid any risk, apply additional means to immobilise the platform (eg. by additionally securing the original platform lock, by means of ratchet straps,...). Signal the hazard in a clearly visible way.
- To re-open the lift, use again an external device (forklift, gantry crane,...) for additional support. Or repair the breakdown first, bring the hydraulic system under pressure, and release the platform locks after satisfactory testing of the lift functions.

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
## 6) OPERATION OF THE LIFT

### §6.1 - Introduction to the control units

- The lifts should be used by means of the original control unit only.

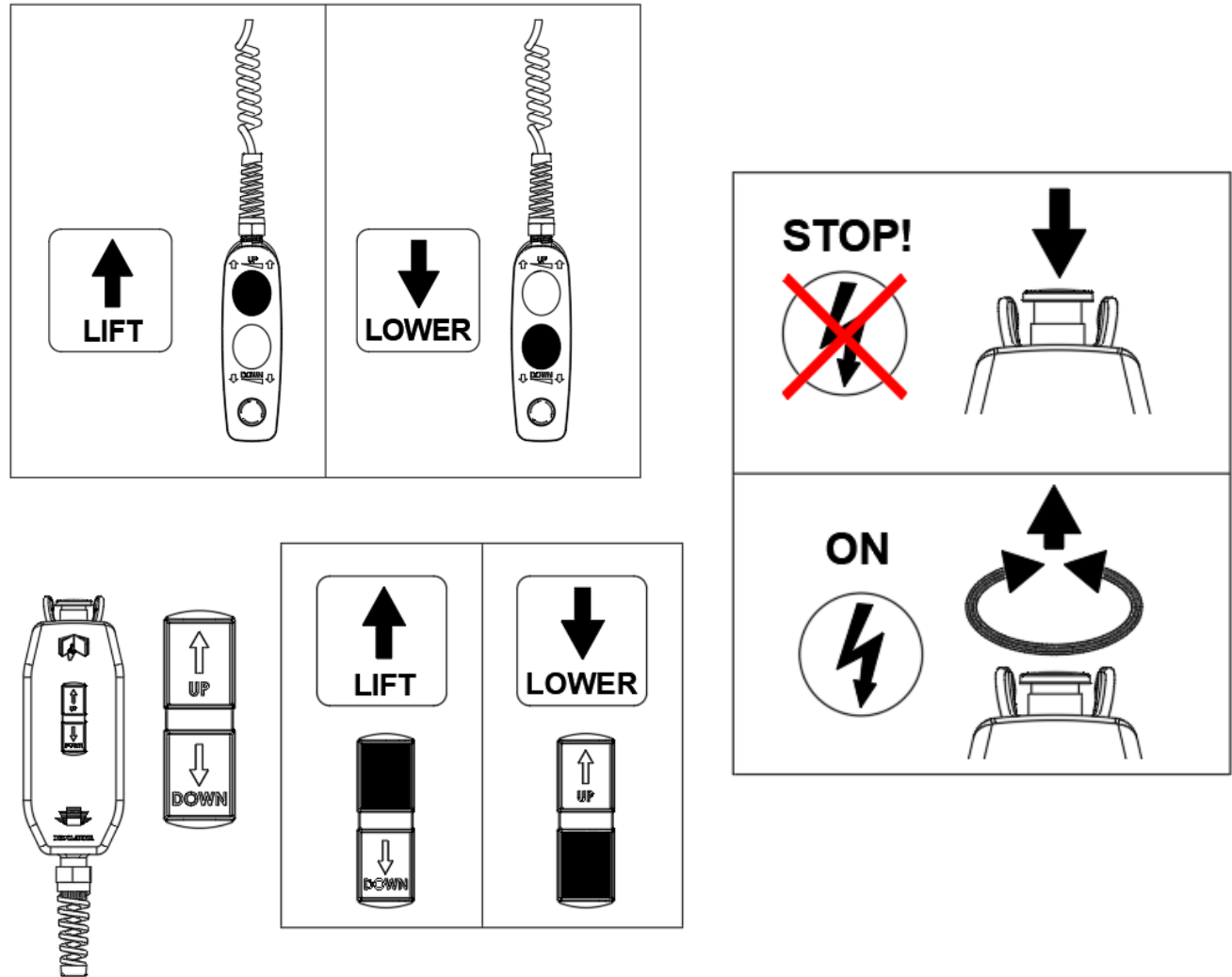Fig. 6.1.1

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§6.2 - Operating instructions

| Operating instructions for DH-RB 05.02.... series |   |
|---|---|
| Fig.  | Opening the platform  |
| 6.2.1   | Consult the user's manual before getting started. Follow all safety instructions. |
| 6.2.2   | Switch on the electrical power to the lift.                                       |
| 6.2.3   | Ensure the emergency button is disengaged   |
| 6.2.4   | Press the DOWN button to lower the lift.  |
| 6.2.5   | Fold open the platform.   |
| ...   | ...   |

Fig. 6.2.1

Fig. 6.2.2

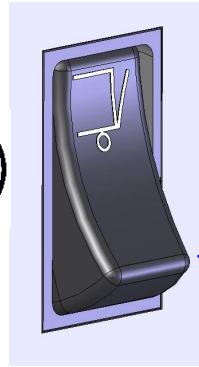
Fig. 6.2.3

Fig. 6.2.4

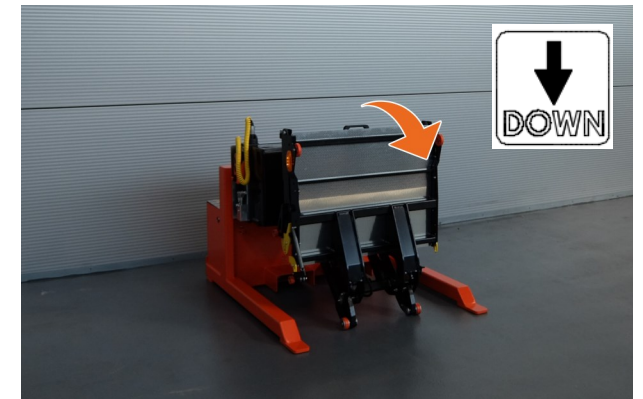
Fig. 6.2.4

Fig. 6.2.5

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|                |  |
|----------------|--|
| Fig.           | Opening the platform   |
| ...            | ...  |
| 6.2.6          | Raise the side safety rails, and lock them in vertical position.   |
| Fig.           | Loading & unloading  |
| -              | Follow the safety instructions and loading diagrams at all times.  |
| 6.2.7          | Press the button DOWN to lower the platform to the ground  |
| 6.2.8          | Press the button UP to lift the platform off the ground to the vehicle floor   |
| 6.2.7<br>6.2.8 | When going DOWN, the bridge plate will automatically raise upon departure of the platform off the loading floor. When going UP, the bridge plate will automatically drop down upon arrival of the platform at the loading floor. |
| 6.2.9          | When going DOWN, the roll-stop will automatically drop down upon arrival of the platform at the ground. When going UP, the roll-stop will automatically raise upon departure of the platform off the ground.                     |
| ...            | ....   |

Fig. 6.2.6


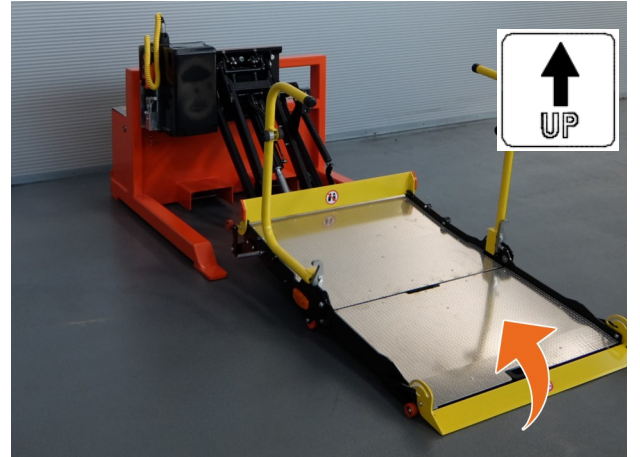
Fig. 6.2.7

Fig. 6.2.8

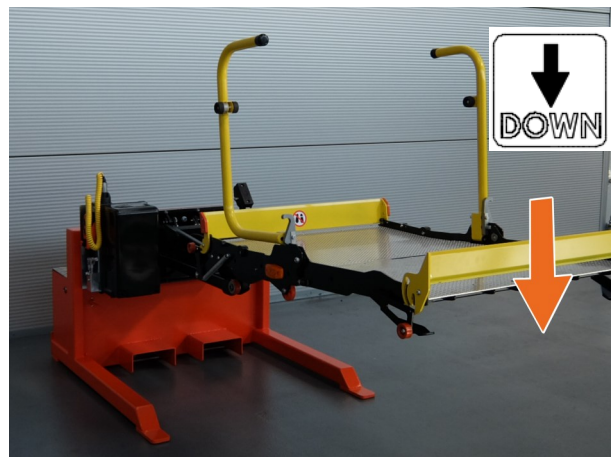Fig. 6.2.9

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|        |   |
|--------|---|
| Fig.   | Closing up in travel position   |
| ...    | ...   |
| 6.2.10 | Fold the side safety rails back onto the platform surface.  |
| 6.2.11 | Fold the first platform part onto the fixed platform part, and fold further till the platform rests onto the lift arms. |
| 6.2.12 | Switch-off the electric power to the lift.  |

Fig. 6.2.10

Fig. 6.2.11


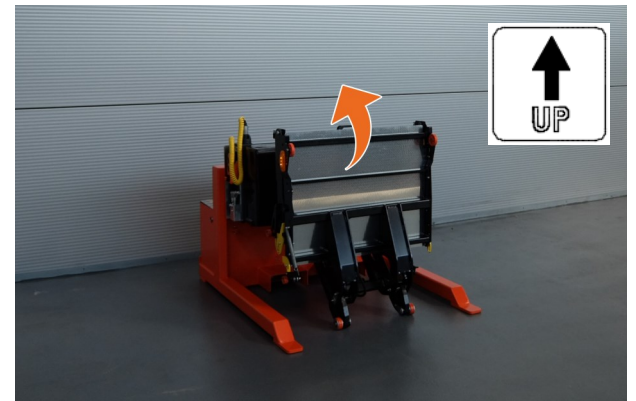
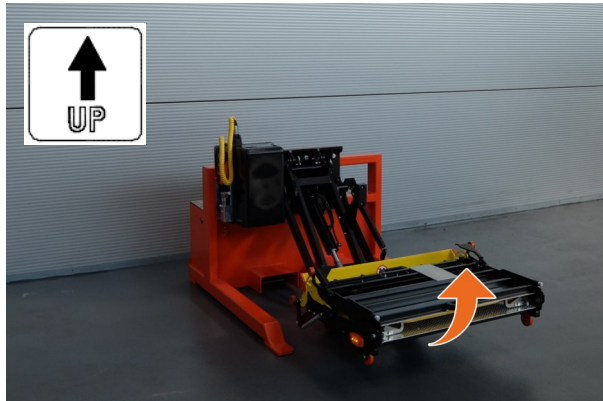
Fig. 6.2.12

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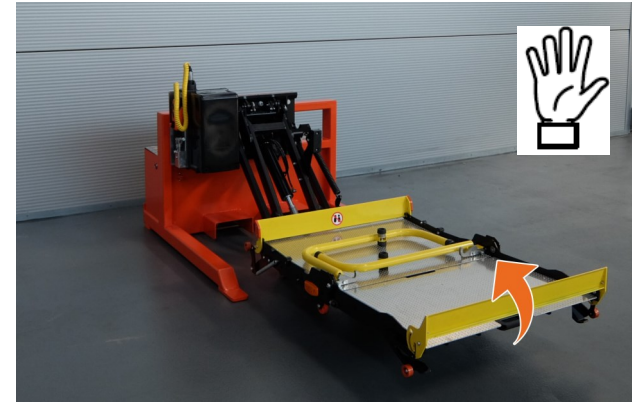### §6.3 - Emergency operation

- In case of electrical failure (battery deficiency, damaged fuse, damaged electrical controls, ...), the various functions of the wheelchair lift can be operated manually as explained below.

#### ⚠ WARNING

- Prior to manipulating the manual emergency controls of the lift, the operator should take full consideration of the safety instructions for use, for repair and maintenance, and the additional precautions outlined below.

- In case of doubt, DO NOT CONTINUE, but contact the national DHOLLANDIA distributor for further help and instructions.
- Negligence can put the technical personnel, the operator and third parties at great risk, and could result in severe personal injury or death.

#### ⚠ WARNING

- To avoid unexpected movements, electric power to lift must be switched off before starting emergency operation
- Always stand clear of the platform area.
- Keep hands and feet clear of pinch points and moving parts

- To proceed, refer to the work sequences above, and replace each electric function by its manual emergency back-up system.

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|   |        |
|---|--------|
|  | Abrir  |
|  | Cerrar |

| Emergency operation to LOWER the platform |  |
|---|--|
| <b>Fig.</b>                               | <b>Manual operation DOWN</b>   |
| 6.3.4                                     | Locate the electrovalve on the power pack  |
| 6.3.4                                     | This valve has an emergency button, concealed under the elastic skin of the nut    |
| 6.3.4                                     | Push-in the elastic skin, and the emergency button under it to LOWER the platform. |
| 6.3.4                                     | The platform stops as soon as the button is released.                              |

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### Emergency operation to LIFT the platform

| Fig.  | Manual operation LIFT  |
|-------|--|
| 6.3.5 | Locate the handle for the hand pump inside the power pack. Fit it over the lever arm of the hand pump. |
| 6.3.5 | Pump the lever up and down to LIFT the platform.   |
| 6.3.5 | The platform stops as soon as the pump action is ended.  |

In order to put any of the emergency systems out of service again, proceed in opposite order

### §6.4 - Electric emergency stop

- The control unit is equipped with an external [ref. OWE005.01] or integrated emergency switch [ref. OWE005.02], enabling to switch off the electric power to the control unit and cause an immediate stop of all electrically actuated movements. [fig. 6.4.1 - 6.4.2].
- Push the emergency button to cut-off the electric power.
- Turn the emergency button to restore electric power.

### 7) WARRANTY TERMS

- The warranty terms on **DHOLLANDIA** lifts may vary from one country to another, and are subject to specific conditions regarding the use, maintenance and inspection of the device.
- Consult your national **DHOLLANDIA** distributor to obtain a copy of the local conditions.

Fig. 6.3.5


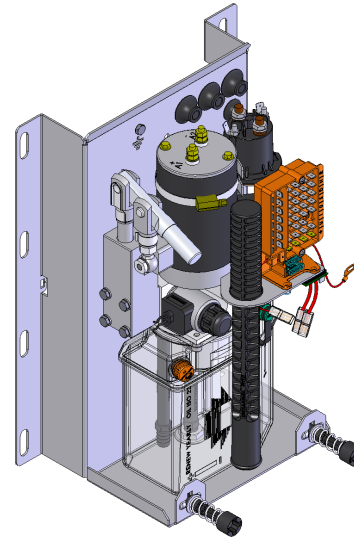
Fig. 6.4.1

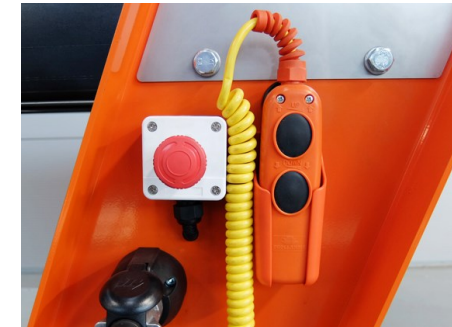Fig. 6.4.2

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**ANNEX 1. SAFETY AND WARNING SIGNS**

| N°  | Description  |
|-----|--|
| 1a  | Overview and keep visual control over the working area of the hydraulic lift at all times  |
| 1b  | General warning sign, used to alert the user to potential hazards. All messages that follow this sign, shall be obeyed to avoid possible harm. |
| 1c  | Entrapment hazard. Keep hands, limbs, loose clothes and long hair away from moving parts.  |
| 1d  | Crushing & sheering hazard. Keep hands away from moving parts  |
| 1e  | Crushing & sheering hazard. Keep feet away from moving parts.  |
| 1f  | Slipping hazard.   |
| 1g  | Tripping hazard.   |
| 1h  | Hazard caused by tilting objects. Always stand clear off the platform work area.   |
| 1i  | Hazard of falling from heights.  |
| 2a  | Contact your national <b>DHOLLANDIA</b> distributor.   |
| 2b  | Download from the DHOLLANDIA.COM website   |
| 2c  | Read the manual or instructions.   |
| 2d  | Hold onto safety rail. Protect yourself from falling off the platform, or vehicle floor.   |
| 2e  | Wear safety gloves.  |
| 2f  | Wear safety shoes.   |
| 2g  | Wear appropriate working clothes, avoid loose-fitting clothes that might be trapped in the moving parts of the lift.                           |
| 2h  | Wear ear protection, eye protection and a safety helmet.   |
| 2i  | Use a safety shield.   |
| ... | ...  |

|   |   |   |   |   |   |   |   |   |
|---|---|---|---|---|---|---|---|---|
| 1a  | 1b  | 1c  | 1d  | 1e  | 1f  | 1g  | 1h  | 1i  |
|  |  |  |  |  |  |  |  |  |
| 2a  | 2b  | 2c  | 2d  | 2e  | 2f  | 2g  | 2h  | 2i  |
|  |  |  |  |   |  |  |  |  |

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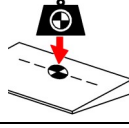
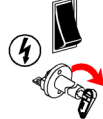
| N° | Description  |
|----|--|
| 3a | General prohibition. Don't do!   |
| 3b | General prohibition. Don't do!   |
| 3c | Only one operator may operate this machine!  |
| 3d | Don't step or stand here!  |
| 4a | Yes do this way. Correct work procedure.   |
| 4b | Yes do this way. Correct work procedure.   |
| 4c | No, don't do this way. Incorrect work procedure.   |
| 4d | Position the load at the applicable centre of gravity or load centre. Follow the load instructions.          |
| 4e | Emergency stop. Will cause an immediate stop of the lift.  |
| 4f | Emergency exit; provision (lever, valve,...) creating an emergency exit.                                     |
| 4g | Unlock. Disengage the mechanical locking system.   |
| 4h | Lock. Engage the mechanical locking system.  |
| 4i | Switch ON the electrical power.  |
| 5a | Switch ON the electrical power to the tail lift via the main battery isolator switch and / or cabin switch.  |
| 5b | Switch OFF the electrical power.   |
| 5c | Switch OFF the electrical power to the tail lift via the main battery isolator switch and / or cabin switch. |

|   |  |   |   |   |   |   |   |   |
|---|--|---|---|---|---|---|---|---|
| 3a  | 3b   | 3c  | 3d  |   |   |   |   |   |
|  |  |  |  |   |   |   |   |   |
| 4a  | 4b   | 4c  | 4d  | 4e  | 4f  | 4g  | 4h  | 4i  |
|  |  |  |  |  |  |  |  |  |
| 5a  | 5b   | 5c  |   |   |   |   |   |   |
|  |  |  |   |   |   |   |   |   |